

OPTOMED

AURORA®




Veterinary Quick Guide


Getting started




1. Check content

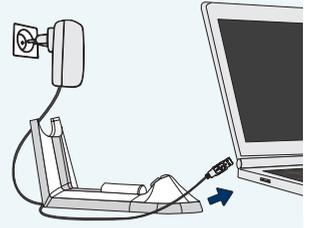


2. Connect

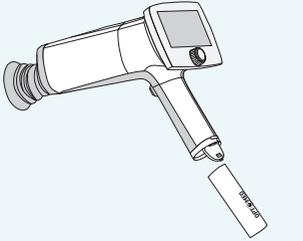

3. Insert battery

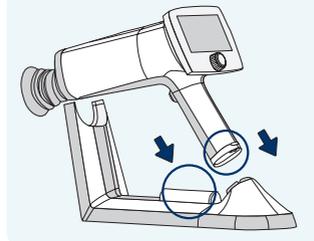

4. Charge batteries

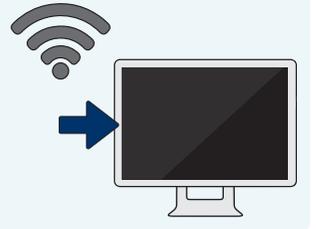

5. For enabling WLAN, install Optomed Client PC software
Instructions on memory stick

User interface



Shutter Button

Battery Cover

Patient

Selected Settings

Power Button

- Short press, power camera on
- Long press, power camera off

Battery Level


WLAN

Back Button


Optoroller

- Access and navigate in menu

Imaging

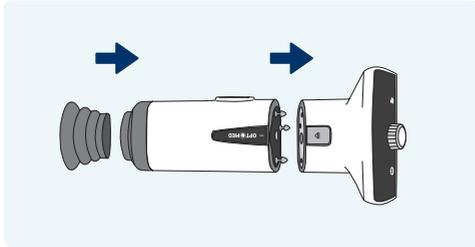


1. Attach optics and optionally an eye cup

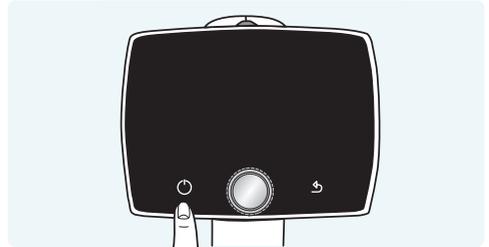


2. Power on



3. Dim room

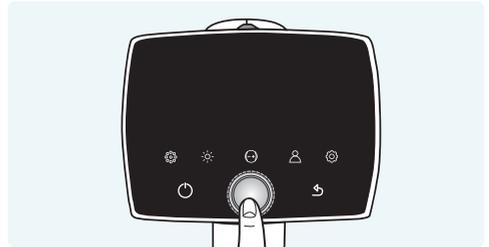

4. Enter menu

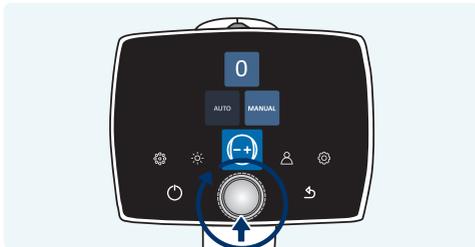


5. Choose focus mode: manual (preferred) or auto



6. Check diopter and modify if needed




7. Choose Manual illumination level. Start with illumination level 0.4 or 0.2 and modify if needed

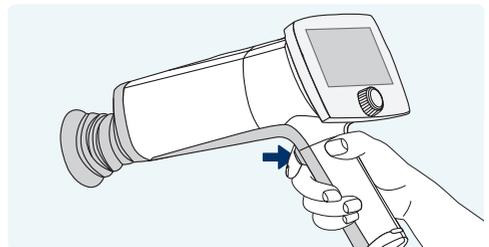

8. Press shutter button for image capture

For veterinary use only.

After imaging

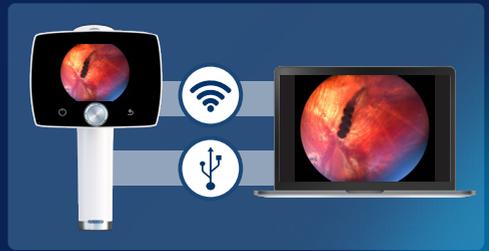

1. Image preview

2. Transfer images using either WLAN or by placing camera to Charging Station



Please check the memory stick provided in sales case for more information about usage and software installations.

Device overview



OPTOMED

For veterinary use only.

www.optomed.com

R1.0
DOC70003114