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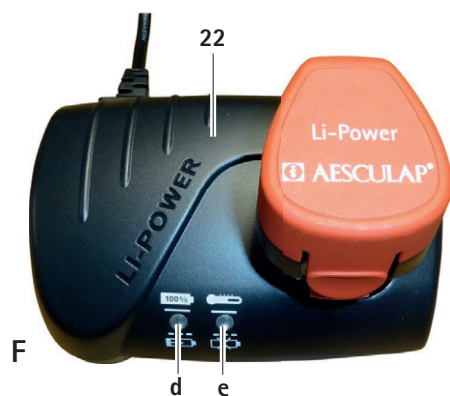
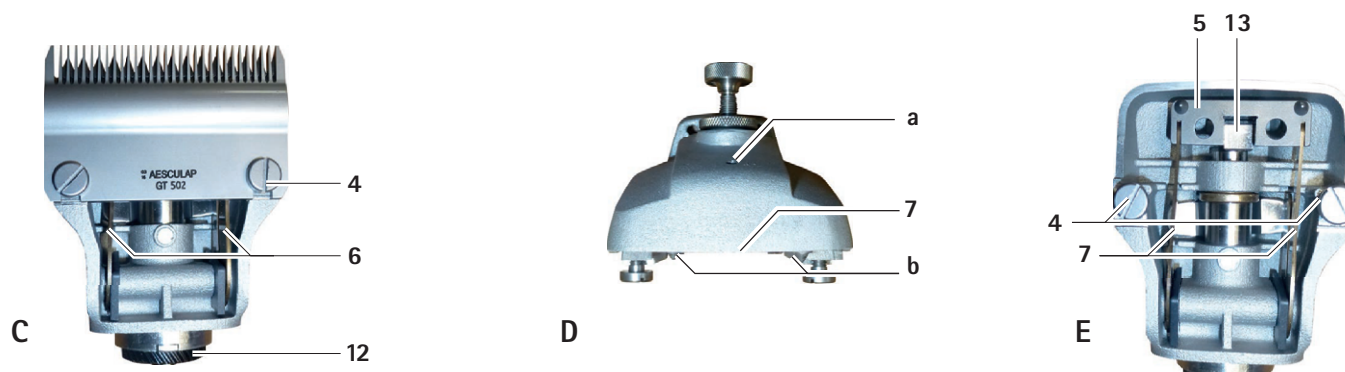
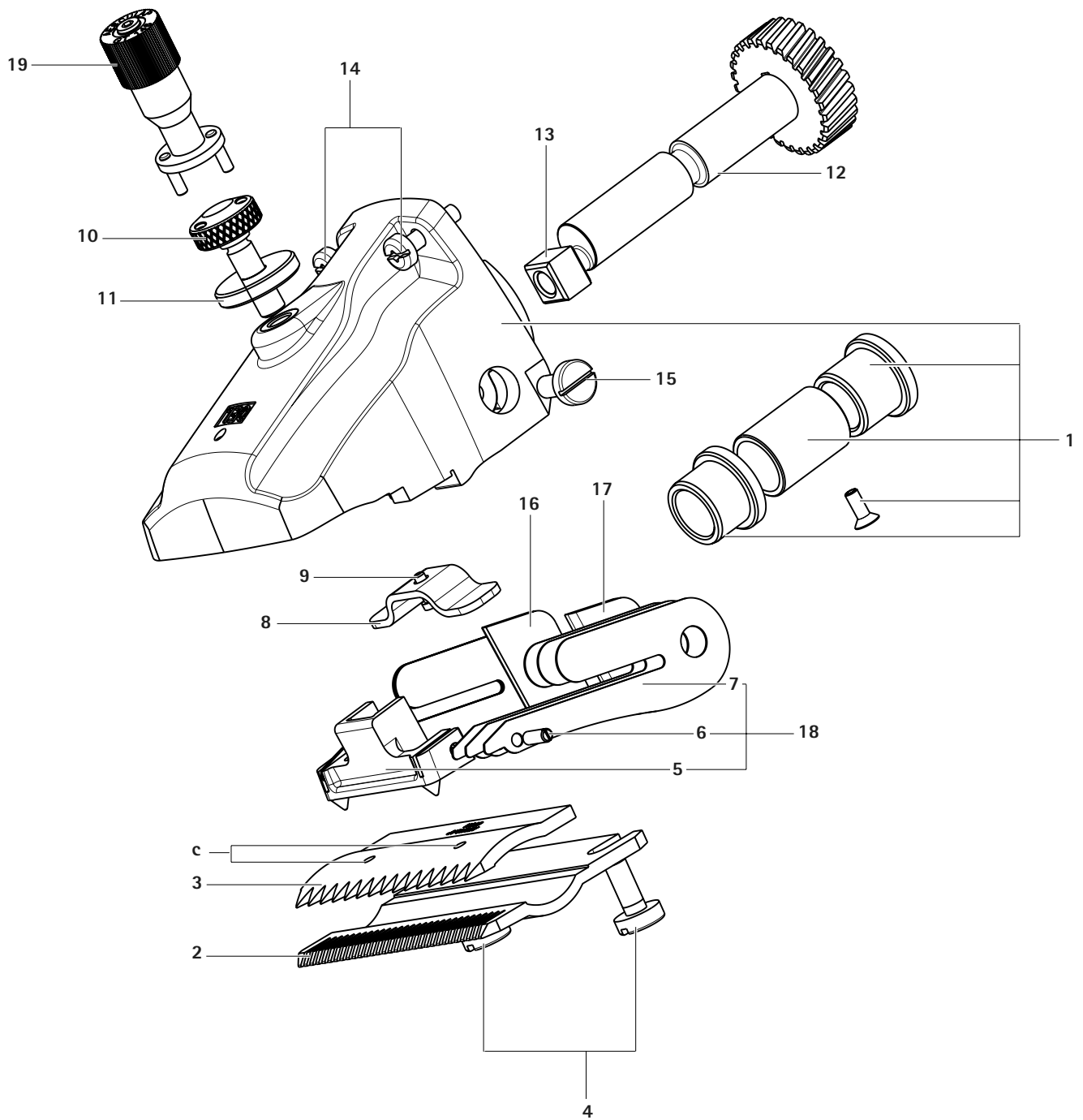
Bonum large animal battery-operated clipping machine

Overview

Item	Art. no.	Name
1	GT647801	1 Mounted cutter head housing
2	GT502 GT504	1 Lower cutter plate
3	GT501 GT503	1 Upper cutter plate (for cows)
	GT505	1 Upper cutter plate (for horses)
4	GT610010	2 Fixing screw
5	GT367201	1 Oscillator
6	GT367202	2 Guide pin
7	TA008295	6 Oscillating spring leaf
8	GT610307	1 Spring bridge
9	GT367207	2 Fixation screw
10	GT640225	1 Adjusting screw
11	GT610405	1 Knurled nut
12	GT640220	1 Shaft with gear wheel
13	GT610304	1 Oscillating block
14	TA010017	2 Screw, cross-head, M4 x 16
15	TA015497	2 Screw
16	GT367803	1 Right distance part
17	GT367802	1 Left distance part
18	GT367804	1 Mounted oscillator
19	GT146	1 Torqui precision adjusting aid
20	GT640249	1 On/off button (with LED)
21	GT641 (3INR19/66-1)	1 Li-ion battery
22	GT643 (LC40)	1 Charger
a	–	1 Lubricating point for battery-operated clipping machine
b	–	1 Tips of the oscillator
c	–	1 Hole in the upper cutter plate
d	–	1 LED, operation indicator
e	–	1 LED, warning indicator
f	–	1 Battery lock
g	–	1 In the UK
–	GT604	1 Special lubricating oil for cutter head and cutter plates
	GT605	1 Gear grease (tube)

Symbols on product

Caution, general warning symbol
Caution, see documentation supplied with the product

Labeling of electrical and electronic devices according to directive 2012/19/EU (WEEE), see Disposal

Contents

1.	Safe handling	3
2.	Product description	3
2.1	Scope of supply	3
2.2	Components required for operation	4
2.3	Intended use	4
2.4	Operating principle	4
3.	Preparation	4
4.	Working with the battery-operated clipping machine, charger and Li-ion battery	4
4.1	System set-up	4
4.2	Function checks	4
4.3	Connecting the charger/Charging the Li-ion battery	4
4.4	Change the Li-ion battery	5
4.5	Handling the cutter head	6
4.6	Storage	7
5.	Cutter plates	7
5.1	Cutter plates compatible with cutter head GT647 Cattle version / GT657 Horse version	7
5.2	Recommended combinations	7
6.	Reprocessing	7
6.1	Cleaning/disinfection	7
6.2	Inspection and checking	8
7.	Maintenance	8
8.	Troubleshooting list	9
8.1	Bonum battery operated clipping machine	9
8.2	Charger and Li-ion battery	9
9.	Technical Service	10
10.	Accessories/Spare parts	10
11.	Technical data	11
11.1	Bonum battery operated clipping machine	11
11.2	Charger GT643 (LC40)	11
11.3	Li-ion battery GT641 (3INR19/66-1)	11
11.4	For EU countries only	11
12.	Disposal	11
12.1	For EU countries only	11
12.2	Li-ion battery	11
13.	Two year guarantee	11

1. Safe handling

Risk of death by electric shock!

- Do not open the product.
- Connect the product only to a grounded power supply.

- Before using the product, verify its functionality and proper condition.
- Store charger and Li-ion batteries at room temperature.
- Keep dry the charger and Li-ion batteries.
- Charge the Li-ion batteries prior to first use.
- To prevent damage caused by improper setup or operation, and to not compromise the manufacturer warranty and liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical facilities in the room in which the device is being used conform to IEC norms.
- Unplug the device by pulling the plug, but never by pulling the power cord.
- Do not operate the product in explosion-hazard areas.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

2. Product description

2.1 Scope of supply

Art. no.	Designation
GT6xx-BL	Bonum large animal clipping machine for cows, blue
GT6xx-RO	Bonum large animal clipping machine for horses, pink
GT6xx-OG-GEA	Bonum large animal clipping machine for cows, green
GT641 (3INR19/66-1)	Li-ion battery
GT643 (LC40)	Charger incl. mains power cord (with device plug)
TA015304	Instructions for use
TA009222	Screwdriver
GT604	Lubricator bottle

2.2 Components required for operation

- Bonum battery operated clipping machine
- Charger
- Mains power cord (with device plug)
- Li-ion battery

2.3 Intended use

Bonum battery operated clipping machine

The Bonum battery-operated clipping machine is used for clipping large animals such as cattle, horses and large dogs.

Charger GT643 (LC40)/Li-ion battery GT641 (3INR19/66-1)

The GT643 (LC40) charger is used to charge the GT641 (3INR19/66-1) lithium ion battery.

2.4 Operating principle

Bonum battery operated clipping machine

The Bonum battery-operated clipping machine is switched on and off using the on/off button 20.

Charger GT643 (LC40)/ Li-ion battery GT641 (3INR19/66-1)

The GT643 (LC40) charger is designed for a mains voltage range of 220 V to 240 V and 50 Hz to 60 Hz.

To ready the GT643 (LC40) charger for operation, connect it to mains power by plugging in the mains plug.

The GT643 (LC40) charger has one charging bay.

Charging starts automatically as soon as the Li-ion battery is inserted in the charging station.

The charging time depends on the charge state and capacity of the Li-ion battery.

Charging principle

The Li-ion batteries are charged by constant current pulses.

The charge state of the Li-ion battery is monitored during charging. Monitoring of the charging curve ensures that the battery is fully charged without overcharging.

When the Li-ion battery is completely charged, the LED, operation indicator **d** lights up continuously.

The Li-ion battery 21 can remain in the charger 22 and is then always ready for operation.

Additionally, the battery temperature and charging time are monitored as well.

Charging time

Charging is stopped as soon as the maximum charging time is reached. The charging time is approx. 53 min.

3. Preparation

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- ▶ When setting up and operating the product, adhere to:
 - national regulations for installation and operation,
 - the national regulations on fire and explosion protection,
 - operating advisories according to IEC-/VDE regulations.

4. Working with the battery-operated clipping machine, charger and Li-ion battery

4.1 System set-up

Connecting the accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

4.2 Function checks

Bonum battery operated clipping machine

- ▶ Carry out a visual inspection.
- ▶ Push Li-ion battery 21 into the machine bay until it clicks into place.

Charger/Li-ion battery

- ▶ Carry out a visual inspection.
- ▶ Prior to connecting the product to mains power:
 - Check the mains power cord for any damage.
 - Inspect product for possible damage (e.g. inspect contacts of the charging bay for bent contacts)
- ▶ Plug in the mains plug at the mains power socket.
- ▶ Place the Li-ion battery 21 in the charging bay of the charger and charge it, see Fig. F.

4.3 Connecting the charger/Charging the Li-ion battery

WARNING

Fire hazard due to battery poles shorted by fluids or metal components!

- ▶ Li-ion battery not inserted

WARNING

Risk of injury due to damaged mains power cord!

- ▶ Check mains power cord for any damage before charging.
- ▶ Arrange for any damaged mains power cord to be replaced by the manufacturer.

WARNING

Risk of injury and material damage due to damaged Li-ion battery!

- ▶ Prior to charging, check the Li-ion battery for any damage.
- ▶ Do not use or charge any damaged Li-ion batteries.

CAUTION

Loss of capacity/performance due to prolonged storage of the discharged Li-ion battery!

- Store Li-ion batteries only in fully charged condition and recharge stored batteries once every month.

CAUTION

Destruction of the product, charging bay or Li-ion battery when inserting the wrong type of battery in the charging bay!

- Insert Li-ion battery only in the appropriate charging bay.

Note

Before starting, ensure that the mains voltage and mains frequency indicated on the nameplate correspond with the specifications of the mains power grid. When connecting the charger, it is recommended that a fault-current circuit breaker of at least type F with a tripping current of max. 30 mA be used.

Note

The charger can be used by children if 8 years or older, as well as by persons with reduced physical, sensory or mental capacities or with lack of experience and knowledge, if these persons are supervised or have been instructed on how to use the device safely, and if they understand the risks associated with the use of the device.

- Do not allow children to play with the device.
- Do not allow children to clean or maintain the device without supervision.

Note

The Li-ion battery heats up slightly during charging.

Note

Disconnect the mains power cord at the mains socket when the charger is not going to be used.

- Only use the Aesculap charger GT643 (LC40) for charging the lithium ion battery.
- Insert the charger 22 power plug.
The LED, operation indicator **d** and the LED, warning indicator **e** light up in succession for approx. 1 second; see figure F.
- Insert the Li-ion battery 21 into the charging bay of the charger 22 up to the stop; see figure F.
The LED, operation indicator **d** blinks and the Li-ion battery 21 is charged.
When the LED, operation indicator **d** is permanently illuminated, the Li-ion battery is completely charged.

4.4 Change the Li-ion battery

- Hold the battery-operated clipping machine **g** with the cutter head downwards so that the lithium ion battery 21 does not fall down when loosened, see Fig. 1.
- Loosen the battery lock **f** with your thumb.

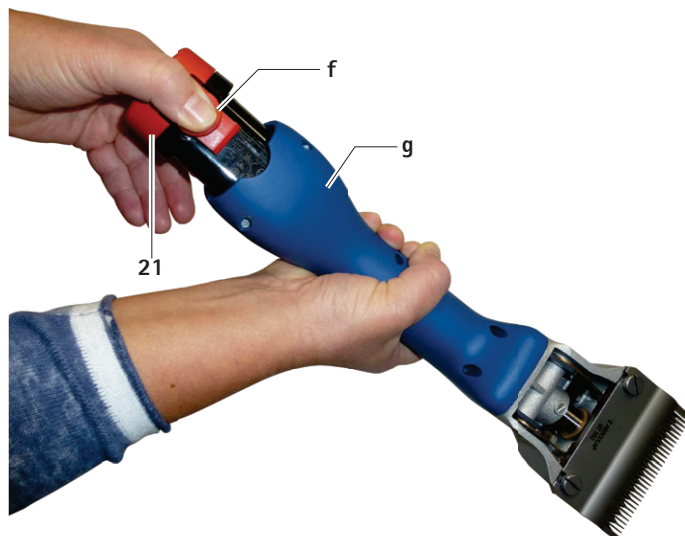
Fig. 1

- Remove the lithium ion battery 21, and push the charged and/or uncharged replacement lithium ion battery GT641 (3INR19/66-1) into the battery-operated clipping machine **g**, see Fig. 2.

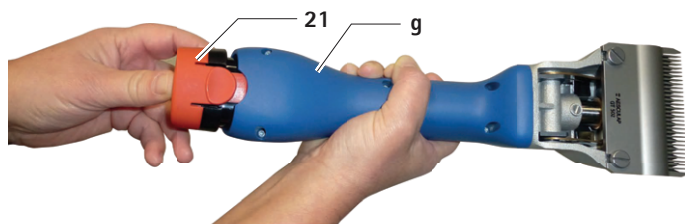
Fig. 2

4.5 Handling the cutter head

Lubricating the shearing head

Damage to the clipping machine or the cutter head due to incorrect lubrication!

- ▶ Only oil the cutter head when the machine is running.
- ▶ Never immerse the cutter head in liquid of any kind!
- ▶ Only use GT604 light lubricating oil.

- ▶ Generously oil the cutter head at the lubrication point **a** before shearing each animal, see Fig. A.
- ▶ Oil spring leaves **7** 2 to 3 times daily with 3 to 4 drops of oil, see Fig. C.

Lubricating the gear wheel

Damage to the clipping machine or the gear wheel due to incorrect lubrication!

- ▶ Use only gear grease GT605.
- ▶ Lubricate the gear wheel **1** to **2** times during the shearing season.

- ▶ Unlock and remove the lithium ion battery **21**.
- ▶ Remove both screws **14**, see figure B.
- ▶ Then turn the cutter head very slightly (about 15°) counterclockwise and remove it from the motor shell.
- ▶ Apply approximately 1 cm of grease on the two opposite points of the gear wheel **12**, see Fig. C.
- ▶ Carefully place the cutter head on the motor. When re-attaching the cutter head, ensure that it is turned 15° counterclockwise in such a way that its lower flange engages with the groove of the motor housing.

Changing the cutter plate

- ▶ Unlock and remove the Li-Ion battery **21**.
- ▶ Loosen the knurled nut **11** and adjusting screw **10** by approximately 2 turns, see Fig. A.
- ▶ Loosen both clamp screws **4**, pull off the lower cutter plate **2** forwards and remove the upper cutter plate **3**, see Fig. A and E.
- ▶ Before inserting the upper cutter plate **3**, ensure that the oscillating block **13** is in the intended recess of the oscillator **5**, see Fig. E.
- ▶ Insert the upper cutter plate **3**. When doing so, ensure that both tips **b** of the oscillator **5** engage precisely in the holes **c** of the upper cutter plate **3**, see Fig. D.
- ▶ Push the lower cutter plate **2** below the heads of the clamping screws **4** all the way on to the cutter head shell and screw both clamping screws **4** tight.
- ▶ Tighten the adjusting screw **10** until both cutter plates are pressing slightly against one another, see Fig. A.

Cleaning the cutter head

- ▶ The cutter head must be cleaned at least after each day of clipping time or if it gets dirty.
- ▶ Unlock and remove the lithium ion battery **21**.
- ▶ Remove both screws **14**, see Fig. B.
- ▶ Then turn the cutter head very slightly (about 15°) counterclockwise and remove it from the motor shell.
- ▶ Remove the cutter plates, see Changing the cutter plate.
- ▶ Thoroughly clean the interior part and individual parts of the cutter head using a brush.
- ▶ Insert the cutter plate and tighten, see Changing the cutter plate.
- ▶ Lubricate the gear wheel, see Lubricating the gear wheel.
- ▶ Carefully place the cutter head on the motor. When re-attaching the cutter head, ensure that it is turned 15° counterclockwise in such a way that its lower flange engages with the groove of the motor housing.
- ▶ Lubricate the cutter head, see Lubricating the shearing head.

Adjust plate pressure with GT146 precision adjusting aid (optionally available)

Note

The manufacturer recommends the GT146 precision adjusting aid for optimal adjustment of the plate pressure.

- ▶ Insert precision adjustment aid GT146 **19** into the 2 holes of the adjusting screw **10** and turn clockwise until there is an audible click.

Note

Knurled screws of older shearing heads that do not have 2 holes can be retrofitted with the GT147 adjusting screw.

Adjusting the plate pressure

Note

The plate pressure must be set with the machine running and without auxiliary tools.

Note

If the plate pressure is too high, this results in the build-up of heat and greater wear to the cutter plates.

- ▶ Loosen the knurled nut **11** and the adjusting screw **10** carefully until the upper cutter plate **3** is no longer pressing against the lower cutter plate **2** (the sound of the motor becomes noticeably higher-pitched); see figure A.
- ▶ Screw in the adjusting screw **10** until you feel slight resistance. From this point, keep turning approximately one full turn and then tighten the knurled nut **11**.

Cleaning the cutter head

- The cutter head must be cleaned at least after each day of clipping time or if it gets dirty.
- Unlock and remove the lithium ion battery **21**.
- Remove both screws **14**, see Fig. B.
- Turn the cutter head very slightly (about 15°) counterclockwise and remove it from the motor shell.
- Remove cutter plates; see Changing cutter plates.
- Thoroughly clean the interior part and individual parts of the cutter head using a brush.
- Insert the cutter plate and tighten, see Changing cutter plates.
- Grease gear wheel, see Greasing gear wheel.
- Carefully place the cutter head on the motor. When re-attaching the cutter head, ensure that it is turned 15° counterclockwise in such a way that its lower flange engages with the groove of the motor housing.
- Lubricate the shearing head, see Lubricating the shearing head

Re-sharpen the cutter plates

- For sharpening the cutting plate, please contact Technical Service, see Technical Service.

4.6 Storage

- Clean the clipping machine and cutter plates thoroughly before storing, see Reprocessing.
- Store the clipping machine in a dry, dark and clean room.

5. Cutter plates

5.1 Cutter plates compatible with cutter head GT647 Cattle version / GT657 Horse version

Art. no.	Designation
GT501	Upper cutter plate with 15 cutting teeth
GT502	Lower cutter plate with 31 cutting teeth 3 mm cutting height
GT503	Upper cutter plate with 17 cutting teeth
GT504	Lower cutter plate with 18 cutting teeth 3 mm cutting height
GT505	Upper cutter plate with 23 cutting teeth
GT506	Lower cutter plate with 23 cutting teeth 3 mm cutting height
GT508	Lower cutter plate with 51 cutting teeth 1 mm cutting height
GT510	Like GT506, but with cutting height of 5 mm
GT511	As GT502, but with cutting height of 1 mm

5.2 Recommended combinations

Combination	Upper/lower plate	Suitable for
closely spaced teeth	GT501/GT502	Cattle and horses (normal configuration)
	GT501/GT511	Industrial shearing
	GT505/GT502	Horses and cattle – smooth standard shearing
	GT505/GT508	Horses, cattle (udder shearing) and industrial shearing
medium spaced teeth	GT505/GT511	Horses and cattle – for very fine, smooth shearing
	GT505/GT506	Cattle, dogs and goats
	GT505/GT510	Dogs and goats
widely spaced teeth	GT503/GT504	Cattle, sheep and thin-wooled sheep

6. Reprocessing

6.1 Cleaning/disinfection

Risk of electric shock and fire hazard!

- Before cleaning:
 - Remove the mains plug from the charger.
 - Unlock the Li-ion battery and remove from the battery-operated clipping machine.
- Do not use flammable or explosive cleaning or disinfecting solutions.
- Ensure that no fluids will penetrate the product.

Damage to, or destruction of the product caused by mechanical cleaning/disinfection!

- Only clean and disinfect the product manually.
- Do not sterilize the product under any circumstances.

Damage to the product due to inappropriate cleaning/disinfecting agents!

- ▶ **Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.**
- ▶ **Never immerse the product in water or detergent.**
- ▶ **Do not introduce any fluid into the charging bays.**
- ▶ **Clean the contacts in the charging bay, applying extreme care.**

- ▶ Wipe the shell of the product with a lint-free cloth moistened with a commercial disinfectant for wipe disinfection.
- ▶ If necessary, after the appropriate exposure time, wipe off any residues of the cleaning/disinfecting agent with a lint-free cloth moistened with clean water.
- ▶ Dry the product with a clean, lint-free cloth.
- ▶ Clean the contacts in the charging bay with isopropanol or ethyl alcohol and a cotton swab. Do not use corrosive chemicals.
- ▶ If necessary, repeat the cleaning/disinfecting process.

6.2 Inspection and checking

- ▶ Allow the product to cool down to room temperature.
- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- ▶ Check the product for any damage, abnormal running noise, overheating or excessive vibration.
- ▶ Inspect the shearing head for broken, damaged or blunt cutting edges.
- ▶ Set aside the product if it is damaged.

7. Maintenance

To ensure reliable operation, Aesculap recommends servicing the product **at least once a year**.

8. Troubleshooting list

8.1 Bonum battery operated clipping machine

Malfunction	Cause	Remedy
The battery-operated clipping machine does not start	Li-ion battery not loaded	Load Li-ion battery
	Li-ion battery defect	Insert new Li-ion battery
	Li-ion battery not inserted	Insert the Li-ion battery
The battery-operated clipping machine pauses during operation	Li-ion battery deactivated through protective circuit	Switch the battery-operated clipping machine off and on again
	Li-ion battery is empty	Insert new Li-ion battery

8.2 Charger and Li-ion battery

Malfunction	Detection	Cause	Remedy
Charger not working	LED, operation indicator d does not illuminate	Mains power cord not plugged in	Plug in the mains connection line at the mains power socket.
	Li-ion battery not loaded	Mains power cord defective	Send charger to manufacturer for repair; see Technical Service
		Li-ion battery defect	Send Li-ion battery to manufacturer for repair, see Technical Service
Li-ion battery cannot be loaded	Li-ion battery inserted, LED, operation indicator d does not light up	Charger contacts dirty	Clean the contacts in the charging bay, see Reprocessing
	–	Charger contacts damaged	Send charger to manufacturer for repair, see Technical Service
	–	Li-ion battery defect	Send Li-ion battery to manufacturer for repair, see Technical Service
	–	Charger defect	Send charger to manufacturer for repair, see Technical Service
	Li-ion battery inserted, LED, warning indicator e lights up continuously	Too high a temperature (>50°) is measured in the Li-ion battery during the charging process	Remove the Li-ion battery from the charging bay, leave to cool and repeat the charging process
			If the problem occurs again, have Li-ion battery inspected by manufacturer, see Technical Service
	Li-ion battery inserted, LED, warning indicator e blinking	Battery charging error, foreign object on contact system or Li-ion battery defective	Remove the Li-ion battery from the charging bay, clean contact system and repeat charging process If the problem occurs again, have Li-ion battery inspected by manufacturer, see Technical Service

9. Technical Service

Risk of injury and/or malfunction!
► Do not modify the product.

Modifications carried out on this technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Albert Kerbl GmbH

Felizenzell 9

84428 Buchbach / Germany

Phone: +49 (0) 8086 933-100

Fax: +49 (0) 8086 933-500

Email: info@kerbl.com

www.kerbl.com

Other service addresses can be obtained from the address indicated above.

10. Accessories/Spare parts

Art. no.	Designation	Design	Li-ion battery	Complete charging station	Accessories
GT644-BL	Bonum Cattle version / Set with 1 battery, blue	Europe, except Great Britain	GT641 (31NR19/66-1)	GT643 (LC40)	GT604 Special lubricating oil for cutter plates and cutter head
GT644-OG-GEA	Bonum Cattle version / Set with 1 battery, green				
GT646-BL	Bonum Cattle version / Set with 2 batteries, blue				GT605 Gear grease (tube)
GT646-OG-GEA	Bonum Cattle version / Set with 2 batteries, green				
GT654-RO	Bonum Horse version / Set with 1 battery, pink				
GT656-RO	Bonum Horse version / Set with 2 batteries, pink				
GT654-BL	Bonum Horse version / Set with 1 battery, blue				
GT656-BL	Bonum Cattle version / Set with 2 batteries, blue				
GT644G-BL	Bonum Cattle version / Set with 1 battery, blue	Great Britain		GT643G (LC40)	
GT646G-BL	Bonum Cattle version / Set with 2 batteries, blue				
GT654G-RO	Bonum Horse version / Set with 1 battery, pink				
GT656G-RO	Bonum Horse version / Set with 2 batteries, pink				
GT654G-BL	Bonum Horse version / Set with 1 battery, blue				
GT656G-BL	Bonum Cattle version / Set with 2 batteries, blue				
GT644K-BL	Bonum Cattle version / Set with 1 battery, blue	USA		GT643K (LC40)	
GT646K-BL	Bonum Cattle version / Set with 2 batteries, blue				
GT654K-RO	Bonum Horse version / Set with 1 battery, pink				
GT656K-RO	Bonum Horse version / Set with 2 batteries, pink				
GT654K-BL	Bonum Horse version / Set with 1 battery, blue				
GT656K-BL	Bonum Cattle version / Set with 2 batteries, blue				
GT644A-BL	Bonum Cattle version / Set with 1 battery, blue	Australia		GT643A (LC40)	
GT646A-BL	Bonum Cattle version / Set with 2 batteries, blue				
GT654A-RO	Bonum Horse version / Set with 1 battery, pink				
GT656A-RO	Bonum Horse version / Set with 2 batteries, pink				
GT654A-BL	Bonum Horse version / Set with 1 battery, blue				
GT656A-BL	Bonum Cattle version / Set with 2 batteries, blue				

11. Technical data

11.1 Bonum battery operated clipping machine

Stroke rate	max. 2 500 rpm
Running time with a Li-ion battery	approx. 70 min to 80 min
Nominal voltage	max. 10.8 V
Capacity	max. 2 A
Weight (with shearing head and battery)	950 g
Application temperature range	0 ° to 40 °

11.2 Charger GT643 (LC40)

Voltage range (current consumption)	220 V to 240 V
Frequency	50 Hz to 60 Hz
Charging voltage/Output voltage	max. 10.8 V
Charging current/Output current	max. 2.3 A
Weight (with cable and power supply unit)	360 g

11.3 Li-Ion battery GT641 (3INR19/66-1)

Cell type	Li
Nominal voltage	10.8 V
Capacity	2 Ah
Charging time	approx. 53 min
Weight	190 g

11.4 For EU countries only

We hereby declare, under sole responsibility, that this product is in conformity with the following norms or norm-defining documents:

2004/108/EC EMC Directive

DIN EN 60335-1

DIN EN 60335-2-8

Safety of household and similar electrical appliances

– general requirements

– for hair shearing machines

Aesculap Suhl GmbH

12. Disposal

Electric tools, accessories and packaging should be handed over for environmentally responsible recycling.

12.1 For EU countries only

Do not throw electric tools in the domestic waste! According to European Directive 2012/19/EU on old electrical and electronic appliances and its implementation in national law, electrical tools that are no longer suitable for use must be separately collected and sent for environmentally responsible recycling.

12.2 Li-ion battery

Do not dispose of Li-ion batteries in domestic waste, fire or water. Defective or used Li-ion batteries must be collected, recycled or disposed of in an environmentally-friendly way according to Directive 2006/66/EC.

13. Two year guarantee

Dear Customer,

Many thanks for choosing one of our products.

For decades, the name Aesculap has stood for proven quality and first-class service. We are known to provide innovative, high-performance products for our customers. Aesculap manufactures high-quality devices and can assure you that it uses high-quality materials and careful manufacturing processes. We would like to point out to you that we are not liable for defects in our products, if these are caused by improper use, normal wear or are caused by the product being improperly used or if inappropriate or not properly functioning third-party products are used.

Parts that are typically subject to wear, and for which we are therefore not liable, are the battery and shearing head. Material defects that do not affect or only negligibly affect the value or function of the product are also excluded from our warranty.

If we are liable under our warranty, we reserve the right to repair or replace the product.

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