

## Coarse Filter



Item No. E21318005

### How to install the Coarse Filter (Item No. E21318005) on the OxyVet IV Oxygen Concentrator (Item No. 213180)

1. Open and remove the filter cover on the back of your OxyVet IV



2. Secure the coarse filter behind the four little plastic pegs on the inside of the filter cover

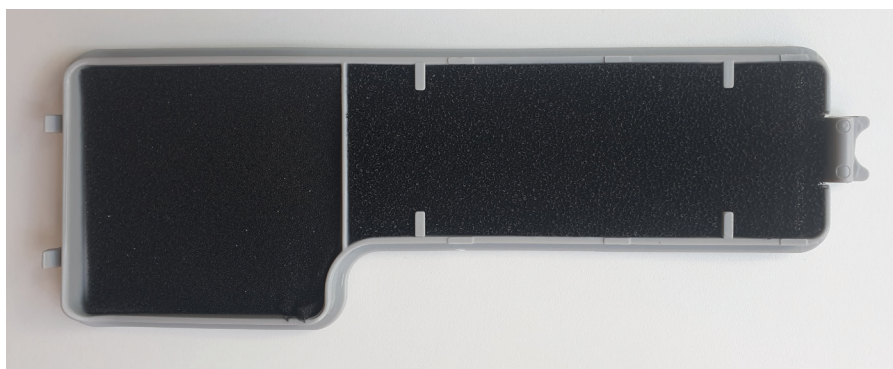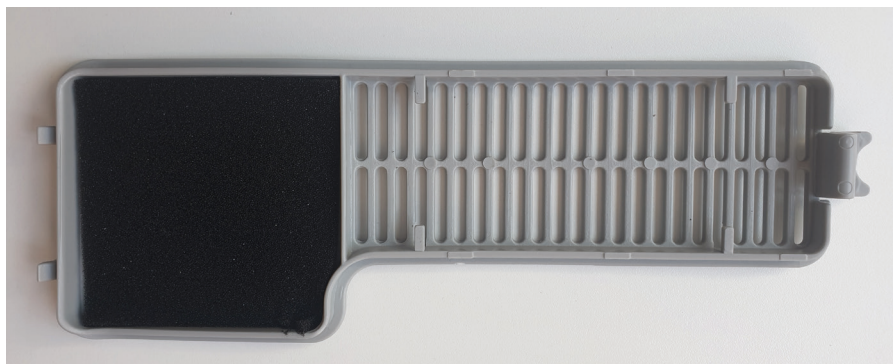

3. Place the filter cover on the back of your OxyVet IV

The Coarse Filter should be cleaned regularly. For cleaning, please remove it, rinse it thoroughly with warm tap water and towel dry thoroughly. The filter should be completely dry before reinstalling.