

EICKVIEW 60S / 60 / 70 / 100E / 150E VIDEOENDOSCOPES

CLEANING INSTRUCTIONS



Item no. 306009 / 306011 / 306010 / 306166 / 306164

TELEPHONE +49 7461 96 580 0

www.eickemeyer.com

veterinary technology for life
Eickemeyer®

CONTENT

Preparation	4
Step 1	4
Step 2	5
Step 3	5
Step 4	6
Step 5	6
Step 6	7
Step 7	8
Step 8	8
Step 9	9

PREPARATION

Cleaning should only be undertaken by authorised trained staff. All safety regulations should be observed (gloves, protective glasses, masks, clothing ...).

STEP 1

1. Please disconnect monitor unit and Luer adapter from endoscope.



Luer adapters with different design

- Immediately after use, remove any heavy soiling (secretions etc.) from the outer casing, shaft and insertion tube of the videoendoscope with a disposable cloth moistened with disinfectant solution or alcohol (max. 70 %).



STEP 2

- Please carry out leakage/pressure test. This test is the most important part of the cleaning procedure.
- Connect the endoscopes to the tester as shown in the picture. Then pump tester till 120 mmHg.
- Now the needle of the pressure tester should not drop by more than 2 mmHg within 5 min, that means pressure should be kept constantly.

If pressure test was succesful you can release air. Afterwards you can continue with cleaning procedure.



Attention!

If pressure drops by a higher value, device is not tight anymore.

Please contact EICKEMEYER® Service immediately. Do not proceed with cleaning procedure.



STEP 3


Please use a suitable single use syringe and clean the working channel properly, 5 – 10 times. Only use disinfectant solutions which are specially accounted for flexible endoscopes.

e. g. GIGASEPT FF from EICKEMEYER® (Item no. 307315)



STEP 4


- Please use supplied cleaning brush to remove large dirt particles from working channel.
- Please ensure that the distal end is in a straight position, otherwise endoscopes can be damaged.

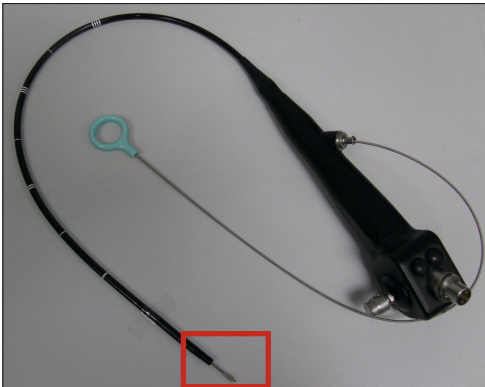

Straight distal end

STEP 5

- Please use solution for another time to clean the working channel, 5 – 10 times.
- Clean distal end with a cotton bud dipped in alcohol (max. 70 %).

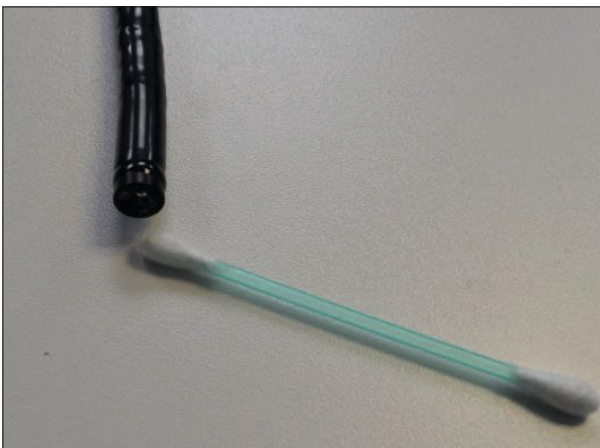

STEP 6

- Now immerse/soak complete endoscope (without monitor unit) in a suitable disinfectant solution.
- Please use disinfectant solution as described in the manual of the solution.
- Luer adapter and cleaning brush can be also immersed/soaked in the solution.



Attention!

Please take care that the black cap is NOT mounted on the endoscope.



STEP 7

- Remove the endoscope from disinfectant solution.
- Dry endoscope with a towel.
- Please remove remaining liquid from all channels, preferably with sterile compressed air (max. 0.5 bar).



STEP 8

- Please hang up endoscope for complete drying on a suitable holding device (2 – 3 days).
- Please keep in mind that the transport case is only for transportation, not for storage.



STEP 9

Please clean all electronical parts like monitor and cables with a disposable cloth moistened with disinfectant or alcohol (max. 70 %).



NOTES

GERMANY

EICKEMEYER KG
Eltastraße 8
78532 Tuttlingen
T +49 7461 96 580 0
F +49 7461 96 580 90
info@eickemeyer.de
www.eickemeyer.de

POLAND

EICKEMEYER Sp. z o.o.
Al. Jana Pawła II 27
00-867 Warszawa
T +48 22 185 55 76
F +48 22 185 59 40
info@eickemeyer.pl
www.eickemeyer.pl

SWITZERLAND

EICKEMEYER AG
Sandgrube 29
9050 Appenzell
T +41 71 788 23 13
F +41 71 788 23 14
info@eickemeyer.ch
www.eickemeyer.ch

DENMARK

EICKEMEYER ApS
Lysbjergvej 6, Hammelev
6500 Vojens
T +45 7020 5019
info@eickemeyer.dk
www.eickemeyer.dk

ITALY

EICKEMEYER S.R.L.
Via G. Verdi 8
65015 Montesilvano (PE)
T +39 0859 35 4078
F +39 0859 35 9471
info@eickemeyer.it
www.eickemeyer.it

UNITED KINGDOM

EICKEMEYER Ltd.
3 Windmill Business Village
Brooklands Close
Sunbury-on-Thames
Surrey, TW16 7DY
T +44 20 8891 2007
F +44 20 8891 2686
info@eickemeyer.co.uk
www.eickemeyer.co.uk

NETHERLANDS

EICKEMEYER B.V.
Bedrijventerrein Pavijen-West
Bellweg 44
4104 BJ Culemborg
T +31 345 58 9400
info@eickemeyer.nl
www.eickemeyer.nl

CANADA

EICKEMEYER Inc.
617 Douro Street, Suite #205
Stratford, Ont. Canada
N5A 0B5
T +1 519 273 5558
F +1 519 271 7114
info@eickemeyervet.ca
www.eickemeyervet.ca