

Quick Guide

Cleaning Instructions for Protective Aprons

Use hanger when not in use.

TO CLEAN

Use a mild soap solution.

DO NOT

launder, autoclave or use detergent.

MAY USE

alcohol to remove stubborn stains.

Apron should be x-rayed periodically to check for cracks in the x-ray protective lining.

EICKEMEYER®

Eltastraße 8 | 78532 Tuttlingen | Germany

T +49 7461 96 580 0 | F +49 7461 96 580 90

E export@eickemeyer.com | www.eickemeyer.com

