


User Manual

Before connecting the device to power, we strongly suggest you to follow the description carefully. The device is connected to the circuit 110 V and will operate with the foot switch.

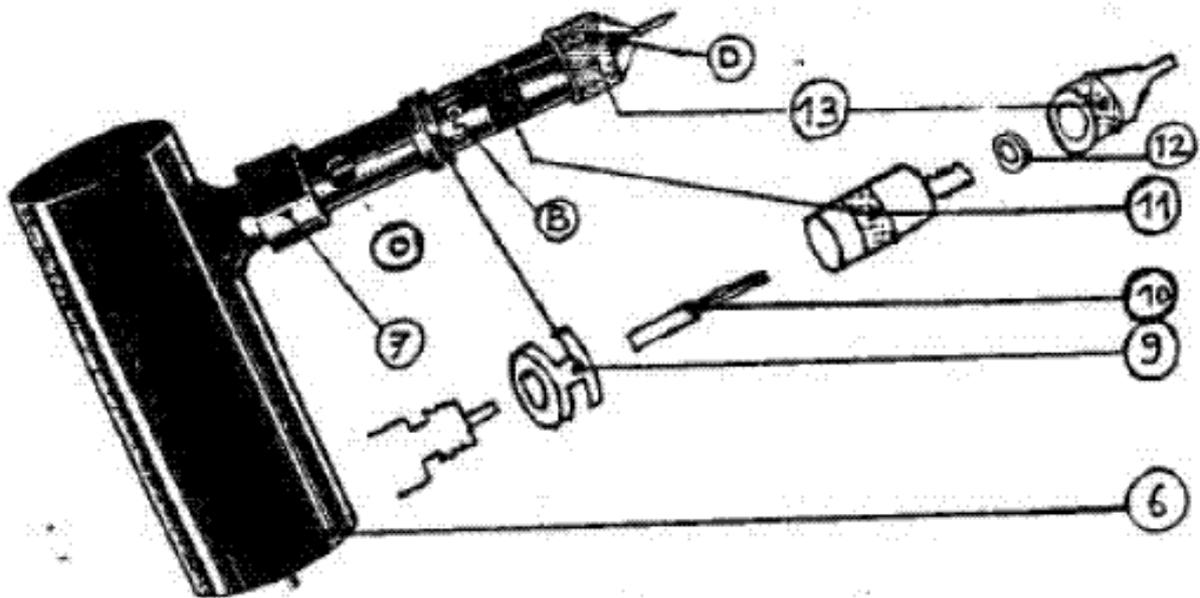

Filling the ink and controlling the flow :

Hold the marker pen with your thumb and index finger at the black ribbing (11) and turn the sleeve clockwise until you notice the resistance . After that, drop a few drops ink by a rubber sleeve Give (7) closed opening (0). It is necessary to keep down the signing pen doing this. then close the opening with the rubber sleeve.

The device will be ready by holding it vertically and opening the protective sleeve by turning left, wait until the ink appears.

Important: If the device contains ink, the needle must always keep down.

The needle length can be regulated by turning the signing pen to the right or left. One turn results in a length difference of 1 mm. The Mark (B) on the marker pen allows precise regulation. The ink flow can be controlled by rotating the protective cap 13.

After using take off the protecting sleeve and the front part of the signing pen (13), so the excess ink can run out. **Please do not leave ink in it.** The complete cleaning can be done by removal (see below). Please make sure that the needles does not stick out when the unit is retired. This is reached using the foot switch until the needle is in the position favored.

disassembly :

To replace the cluster of needles (10), you have to unfasten the protective sleeve (13) and after that to unscrew the signing pen . The whole thing will then forward pulled out. Then you are able to pull out the cluster of needles (it is plugged and not screwed). When assembled please process in reverse order.

Please note: Never screw the signing pen with screwed protecting sleeve . Otherwise the needle might not pass the white sealing ring. The rubber ring (9), the sealing ring (12) and the Rubber sleeve (7) can also be replaced.

Advice of use:

Shave well and rub lightly with Vaseline the area you want to tattoo. The Cluster of needles should ride out at least about 1.5 mm, so that the needles can penetrate about 1 mm into the skin. Tight the skin and hold the tattoo unit vertically like a pencil. Write continuously **slow**, do not dot. Wipe the unclean ink sites on the skin. The tattoo should appear in black / green. If the bar is red, the skin was not tightened sufficiently or the depth of the needle is too small. The badly tattooed part have to be edit again after you have changed the length of the needles. If a needle is blunt, then you have to replace the cluster of needles. A blunt needle is pushed back by the skin and prevents the penetration of the other needles. For smaller animals, it is possible to draw a very fine line, for that you can break the 4th needle at the base and you will write with only three needles.