CLIPPER SPRAY

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GB Effective spray. Disinfects, lubricates, cleans and cools simultaneously. Prevents fungicidal and virucidal action. Removes hair and dirt. Works like a lubrication which guarantees an even and effective cutting. Regular use of clipper spray will keep the blades sharp and guarantee a longer working life. Hold the can vertical at a distance of approx. 15–20 cm. Spray the blades thoroughly after each use. Allow to dry.

Danger! Extremely flammable aerosol. (H222) Causes skin irritation. (H315) Causes serious eye irritation. (H319) May cause drowsiness or dizziness. (H336) Pressurised container: May burst if heated. (H229) Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F. Pressurised container: Do not pierce or burn, even after use. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. If medical advice is needed, have product container or label at hand. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to an approved waste disposal plant. Read label before use. Keep out of reach of children. Contains: Hydrotreated heavy Naphtha (petroleum), (benzen<0,1%). **For veterinary use only.**

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