# CLINIC 6

Cryosurgery made simple

## Medical user Manual

---

*freezpen*<sup>®</sup> 8g-16g

## Table of contents

---

1 - User manual	4
2 - Manuel d'utilisateur	14
3 - Benutzerhandbuch	24
4 - Gebruikershandleiding	34
5 - Manual del usuario	44
6 - Manual do Utilizador	54

## User manual

---

The Freezpen is a non-invasive, non-sterile, reusable medical device for the cryogenic treatment of the following benign skin lesions: acrochordon, actinic keratosis and cheilitis, capillary hemangioma, cutaneous leishmaniasis, discoid lupus erythematosus; filiform, flat, vulgar, plantar and anogenital warts, rosacea, lichen sclerosus, granuloma annulare; molluscum contagiosum; seborrheic keratosis and viral lesions. Lesions are treated by applying an extremely precise flow of nitrous oxide directly on the lesion.

The lesion and its location must be always diagnosed by a healthcare professional.

H&O Equipments  
Rue des Journaliers 1  
7822 Ghislenghien, Belgium.

+32 68 26 86 06

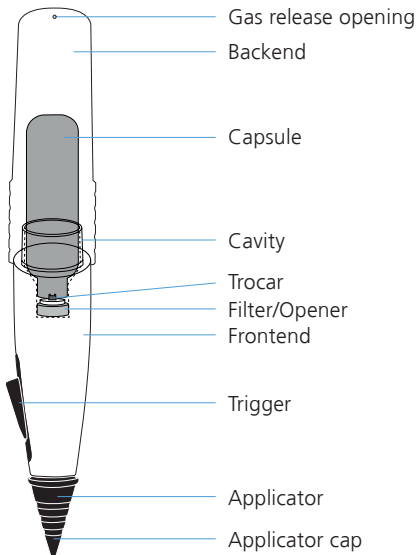
+32 68 28 75 15

info@clinic6.com

www.clinic6.com - www.freezpen.com

Canadian license number 99271

Liability waiver: please read the section 'Liability waiver', page 13.

Manufacturer

Caution, consult accompanying documents

Consult the user manual

Do not expose the capsule to temperatures below -30°C/-22°F and above 50°C/122°F

Keep away from sunlight

Catalogue number

Serial number

Batch number

Use-by date

Notified body nr 1639 SGS BELGIUM

## Important Notice

- When used correctly, the device does not come into contact with the lesion nor the skin.
- When used normally, there is no need to perform any maintenance, cleaning or disinfection.
- Should the applicator touch the patient in case of misuse, the applicator can be cleaned (autoclaved: 134 °C/273 °F - 4 min or 121 °C/250 °F – 20 min) and disinfected (use a non-corrosive disinfecting solution, follow the disinfecting solution manufacturer's recommendations with regards to the concentration and time of application).
- Always handle your Freezpen and its components gently. Never use more than gentle hand force.
- Incorrect placement of the filter/opener will cause irreversible damage to the instrument.
- Leaving seals in the loading cavity while placing a new filter/opener will cause instrument failure.
- When loaded, the instrument is under high pressure. Any manipulation other than normal use may be dangerous.
- Never block the gas release opening at the top of the Freezpen's back end!
- All external parts can be wiped with a cloth soaked in any non-corrosive disinfecting solution. Never immerse the entire instrument in a disinfecting solution.
- Never expose capsules to temperatures over 50°C/122°F. Leave the cap on the device when not in use.
- Never expose cartridges to temperature below -30°C/-22°F.
- The Freezpen should be handled with care. When storing the instrument ensure that the trigger is in the free position.
- Freezpen instruments are intended for professional use only.
- Best results are obtained with the instrument and the capsules at room temperature (21°C/70°F). Lower and higher temperatures will give different values in respect of pressure and the density of the cryogen. Over 25°C/77°F, the quantity of liquefied N<sub>2</sub>O in the Freezpen jet will decrease.
- Empty capsules should be discarded as metal scrap.

## Contraindications

---

- Unstable diabetes
- Skin conditions, e.g. skin tumors, exanthema, open wounds, solar hyperkeratosis
- Unexplained, suspicious liver spots and moles
- Cancer tissues and malignant tumors

## Contraindications related to temporary conditions

---

- Infections accompanied by fever
- Acute chemotherapy or radiotherapy from four weeks before beginning the therapy to four weeks after finishing the therapy
- Pregnancy or breastfeeding
- Cold intolerance
- Vascular insufficiency

## Possible side effects

---

Although cryotherapy is a relatively low-risk procedure, some side effects may occur as a result

of the treatment. They include:

### **Permanent changes in pigmentation.**

Both hypopigmentation (lightening of the skin) and hyperpigmentation (darkening of the skin) may occur after cryotherapy. Both generally last a few months, but can be longer lasting.

**Sensory impairment.** Though rare, damage to nerves is possible, particularly in areas where these are positioned closer to the surface of the skin, such as in the fingers, the wrist, and the area behind the ear. Reports suggest effects will disappear within several months.

**Spattering of the cryogen during spraying, when the end nozzle freezes.** The innovation of Freezpen is the direct application of nitrous oxide under high pressure (49-53 bar for a cartridge temperature of 66-72°F / 19-22°C ). This high pressure jet may cause minor shards of frozen humidity in the air to be blown away in a radius of approximately 30 cm. They will thaw the moment they might touch healthy skin.

**Hair loss.** Hair follicles are easily damaged by

cryosurgery and permanent hair loss is not uncommon.

**Pigmented spots. It is common use to not treat pigmented spots in sunny seasons.** Solar lentigines (SL) are flat, brown, sharply demarcated pigmented lesions that commonly appear as a sign of aging and photodamage on sun-exposed skin.

This user manual is applicable for the following products and accessories.

## Instruments

- **Expected lifetime : minimum 10 years in normal use.**

Freezpen 8g with blue and white applicator

Freezpen 16g with blue, white and green applicator

**REF**

CL-FP-03

CL-FP-19

## Capsule - Expired date on the capsule packaging.

8g N<sub>2</sub>O capsule: 1 carton of 16 boxes each  
containing 21 capsule

8g N<sub>2</sub>O capsule: 1 box of 21 pcs capsule

16g N<sub>2</sub>O capsule: 1 carton of 12 boxes each  
containing 12 capsule

16g N<sub>2</sub>O capsule: 1 box of 12 pcs capsule

CL-N2O-8G-NCARTON

CL-N2O-8G-NBOX

CL-N2O-16G-NCARTON

CL-N2O-16G-NBOX

## Applicators

Blue applicator: for 1-3mm lesion

White applicator: for 2-6mm lesion

Green applicator: for 4-8mm lesion

CL-FP-APPL-00

CL-FP-APPL-01

CL-FP-APPL-02

## Rare side effects

---

- Edemas
- Bleeding
- Pain and fainting
- Insufflation of subcutaneous tissue
- Retraction caused by freezing next to orifices
- Cartilage defect during treatment of the nose or the ear
- Infection
- Conjunctival ectopy due to treatments of the eyelid
- Milia
- Depressed or atrophic scars.

## Steps:

---

### Step 1 | Empty the loading cavity

Release any unused gas from the used capsule by unscrewing the applicator and pressing the button. Then unscrew the capsule cover cap.

Discard the used capsule and check that the cavity is entirely empty.

### Step 2 | Load a new capsule

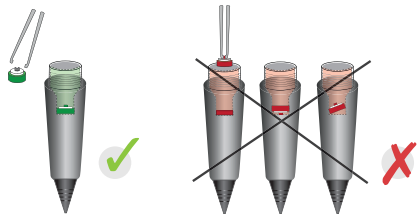
The capsules are delivered in blister packs with an assembled filter and capsule opener included.

Capsules are packed in blister with a filter / opener

Open the blister pack. Take out the filter/opener. Hold the tip of the Freezen pointing downward. Place the filter/opener and make sure that it sits flat at the bottom of the loading cavity.

The trocar of the opener must be pointed upward. Take the capsule and place it in the cavity, on top of the filter/opener device.

Installation of the assembled filter/opener device.

### Step 3 | Puncture the capsule

Keep the front end (tip of the Freezpen) pointing downward. Gently screw the Freezpen back end to the front end until you feel a little resistance. Now do one last twist in a rapid, uninterrupted rotation to puncture the capsule. You might hear a short hissing sound. The shorter the hiss, the better!

### Step 4 | Use your Freezpen

Remove the applicator cap. Push the trigger to start the flow of cryogen. Release the trigger to stop the flow of cryogen. You may notice that after releasing the trigger, the remaining gas in

the applicator is expelled. To switch the applicator, unscrew the current one and screw the other one on the Freezpen.

The table below gives the estimated application with a continuous flow, time depending on the applicator and the cartridge size.

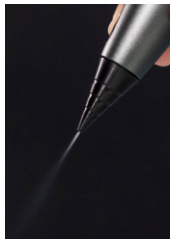
The measurements were performed between 19°C/66°F and 21°C/70°F.

	Freezpen 8g	Freezpen 16g	Lesion size
Blue	135-155s	230-270s	1-3 mm
White	80-100s	145-180s	2-6 mm
Green	<del>X</del>	110-130s	4-8 mm

## Positioning the Freezpen

---

Hold the instrument vertically with the tip pointing downward. Never use the Freezpen at an angle of less than 45° from the vertical position.

Position the Freezpen correctly

## Effective application

---

Before starting treatment consider the pathology, location, size and depth of the lesion, the skin type,

the degree of tanning, the season of the year and other parameters that may influence the result of the treatment.

While using the Freezpen, the applicator tip with the jet of cryogen should be held 0.5 to 3 mm from the lesion. If the tip is held further than 5 mm from the lesion, the liquefied N<sub>2</sub>O will expand into gas by the time it reaches the tissue, which is not effective for achieving therapeutic results.

By approaching the lesion with the jet already flowing you can observe the focal point of the jet on the tissue. The closer you get to the lesion, the more freezing power you will apply to the tissue. It is recommended to keep a greater distance for the smaller and more delicate procedures.

The amount of liquid N<sub>2</sub>O that you wish to apply can be controlled by moving the applicator quickly towards and away from the tissue. Circular and crossing movements have the same effect.

Do not hold the Freezpen applicator still over the lesion. It may flood the area and damage healthy

tissue surrounding the lesion. This can cause blisters and pain.

Condensation of humidity in the air on the frozen working area during the procedure may create an ice layer on top of the lesion. This 'igloo' will prevent the liquid cryogen from working effectively (at  $-89^{\circ}\text{C}/-128^{\circ}\text{F}$ ): it should be removed with the applicator tip by gently wiping the ice.

Protect the area from blown ice shards, although they are relatively harmless.

After thawing, the tissue turns reddish and appears inflamed due to interstitial bleeding. Some edema and/or blistering may occur and cause discomfort for some hours.

Always protect this new-grown skin by using a sunblock for 8 to 20 days after the scab has come off.

## **Duration/depth of the treatment**

Hold the tip of the applicator as close as possible to the lesion. In all literature, the rapid drop

of temperature (thermal shock) is described as a critical criterion. However, always take into account the part of the body being treated. In some parts, skin is thinner and requires more caution. A typical freeze on viral infections may last from 5 to 10 seconds for a small flat wart, and up to 45 seconds for a full-thickness plantar wart. In general, most lesions take about 2 to 30 seconds. The duration of the treatment will depend on the surface area and the thickness of the tissue. Thickness must be estimated by the operator based on clinical experience.

After the first freezing cycle, the tissue should be allowed to thaw for about 30 seconds, followed by a second freezing. Please note that in the second cycle the tissue will freeze faster. This 'freeze-thaw-freeze' technique is recorded in most literature on cryotherapy and gives the best results.

A follow-up visit is recommended to confirm that all the pathological tissue has disappeared or that a follow-up treatment is necessary.

## Patient comfort

---

Inform patients that there may be a pain sensation similar to a ballpoint being pushed onto the skin. Generally, little or no discomfort is experienced during the first few seconds if you avoid healthy tissue. The freezing effect will reach the deepest part of the lesion after a given amount of time depending on the depth of the lesion. From that moment, the patient will experience a pain sensation like a sting. This may be the moment to stop treatment. You may possibly add a few more seconds depending on the patient's comfort level and your clinical evaluation. There might be a little residual stinging for some time after the treatment. Occasionally, a blister might form and persist for a few hours.

For more information visit our websites:  
[www.clinic6.com](http://www.clinic6.com) - [www.freezpen.com](http://www.freezpen.com)

### Liability waiver

Improper use, including excess freezing levels beyond those which are recommended or for an excessive duration, may result in bodily injury to clients/patients or to the operator. Clinic6 sprl and their affiliates, respective directors, officers, shareholders, employees, agents and contractors are not liable or responsible, regardless of whether such liability or responsibility is based on breach of contract, tort, strict liability, breach of warranties, failure of essential purpose, fundamental breach or otherwise, for any death or injury, whether physical or mental, or for any incidental, consequential, indirect, special or punitive damages, arising out of the Freezpen, its design, specifications, possession and use, and treatment procedures, and whether or not any such death, injury, loss, damage(s) result from the negligence, default or error in judgment by Clinic6 sprl, their affiliates, respective directors, officers, shareholders, employees, agents and contractors, and even if advised of the possibility of such damages. You agree to indemnify Clinic6 sprl, their affiliates, respective directors, officers, shareholders, employees, agents and contractors, from and against any and all liability, damages, losses, costs, judgments, fines, penalties and expenses (including legal expenses) of any kind or nature, including, without limitation, incidental, consequential, indirect, special or punitive damages, arising out of claims, demands, actions, causes of action, proceeding or suits, whether in law or in equity, due to any death, injury, loss, damage or damages as herein before referred to.

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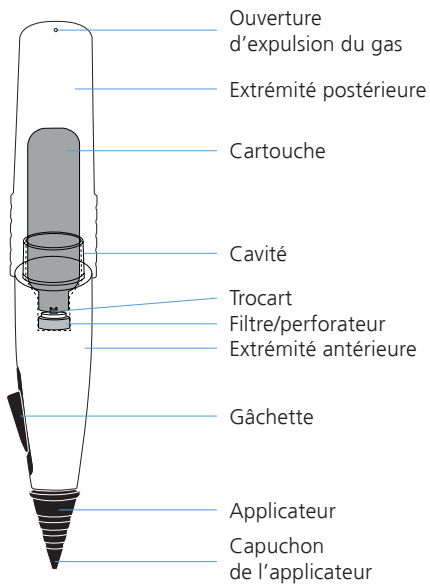
Fabricant

Attention, consultez les documents joints

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