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1. GENERAL INFORMATION

You have purchased one of our quality EICKEMEYER® products. This product has been manufactured according to the best available technology. If looked after according to the guidelines, it should last for many years.

2. INSTRUCTIONS FOR USE

You can raise and lower the table with its built-in electrical footswitch.

3. DAILY INSPECTION

Check daily for

- scratches and dents
- vertical clearance of the table itself
- tightening of screws and nuts.

In the event of damage or failure no attempt should be made to tamper with or make any type of repair, instead please contact the EICKEMEYER® customer service for further information, stating the nature of complaint.

4. TROUBLESHOOTING

- 4.1 The table only moves in one direction (e.g. up but not down): This can be solved by changing the footswitch.
- 4.2 The table makes noises: Please check the lubrication of the table. If necessary, lubricate the table on the sliding surfaces.
- 4.3 Deformities: Check the table for any abnormalities like dents or twisted parts. If so, please contact EICKEMEYER® Support.

5. CLEANING

- 5.1 To reduce the risk of transmission of infectious agents, remove all organic debris (e.g. blood, tissue and other fluids).
- 5.2 For a deep clean, a disinfectant (e.g. Quinticare, 637500) should be applied using disposable paper towels to all surfaces and dry well.
- 5.3 Please be advised that the surface needs to be completely dry before covering the table with a mat.

WARNING: Do not use desinfectants that contain chloride as they can corrode the surface!

6. REPAIR AND MAINTENANCE

Under normal circumstances, no maintenance is required during the lifetime of the equipment. If repairs are required, please contact EICKEMEYER® Support to ensure full integrity of electrical equipment and connections. Warranty and product liability are invalidated where alterations, modifications, or incorrect use of the pillar has occurred.

All internal moving parts are lubricated for life and require no further lubrication.

7. HEALTH AND SAFETY

- 7.1 Table should only be operated by trained staff.
- 7.2 Mobile version: For safety reasons only move the table whilst in lowered position.
- 7.3 Do not get under the table when it is not secured against sudden drop-down.
- 7.4 Mobile version: When moving the table, ensure there is no one in the surrounding area who could get hurt.
- 7.5 Only lift the table whilst it is standing on level ground. Do not use the table on gradient surfaces.
- 7.6 Do not change the setup of the table without the guidance of EICKEMEYER®.
- 7.7 Have the table maintained regularly by a skilled service technician.

In the event of damage or failure no attempt should be made to tamper with or make any type of repair, instead please contact the EICKEMEYER® customer service for further information, stating the nature of issue.

8. CONTACT

Should you need any help or advice on initial use and installation or have any queries regarding the use of the pad please contact your local EICKEMEYER® customer service via telephone or email.

GERMANY

EICKEMEYER KG
Eltastraße 8
78532 Tuttlingen
T +49 7461 96 580 0
F +49 7461 96 580 90
info@eickemeyer.de
www.eickemeyer.de

SWITZERLAND

EICKEMEYER AG
Sandgrube 29
9050 Appenzell
T +41 71 788 23 13
F +41 71 788 23 14
info@eickemeyer.ch
www.eickemeyer.ch

POLAND

EICKEMEYER Sp. z o.o.
Al. Jana Pawła II 27
00-867 Warszawa
T +48 22 185 55 76
F +48 22 185 59 40
info@eickemeyer.pl
www.eickemeyer.pl

ITALY

EICKEMEYER S.R.L.
Via G. Verdi 8
65015 Montesilvano (PE)
T +39 085 935 4078
F +39 085 935 9471
info@eickemeyer.it
www.eickemeyer.it

DENMARK

EICKEMEYER ApS
Solbakken 26, Hammelev
6500 Vojens
T +45 7020 5019
info@eickemeyer.dk
www.eickemeyer.dk

NETHERLANDS

EICKEMEYER B.V.
Doejenburg 203
4021 HR Maurik
T +31 345 58 9400
info@eickemeyer.nl
www.eickemeyer.nl

UNITED KINGDOM

EICKEMEYER Ltd.
3 Windmill Business Village
Brooklands Close
Sunbury-on-Thames
Surrey, TW16 7DY
T +44 20 8891 2007
info@eickemeyer.co.uk
www.eickemeyer.co.uk

CANADA

EICKEMEYER Inc.
617 Douro Street
Suite 205
Stratford, Ont. Canada
N5A 0B5
T +1 519 273 5558
info@eickemeyervet.ca
www.eickemeyercanada.ca