Consult Table

User Manual

6

E

5



# Thank you for choosing ALTRA!

The ALTRA consult table was created by a dedicated team of engineers, veterinarians and designers. Our team would like to say thank you for choosing the ALTRA! You've just invested in cutting-edge technology, great aesthetics and improved work-place ergonomics.

ALTRA is the most advanced consult table on the market. It has a strong steel structure and a medical grade composite worktop. This beautiful surface resembles natural stone and while it is smooth for easy cleaning, it is not slippery, and it is warm to the touch which helps to keep animals calm and relaxed.

Large foot controls allow for smooth height adjustment from any side of the table while standing or seated. Fewer wires mean a safer work environment; and this battery-operated table, which can be easily manoeuvred when and where needed, is wireless.

# Before first use

- Check that the package is not damaged. In case of damage, immediately take photos and add notes to the CMR before signing it. Later claims for transport damages are not accepted by transport companies.
- Always have safety-trained individuals unpack, install and power the table.
- Only lift using the table handles. Lifting the table from any other place may damage the table.
- Keep in mind the basic safety and ergonomic requirements while lifting table from packaging. Be safe! Don't hurt your back!
- Ensure all moving parts are properly fixed and moving free from any obstacles.
- Remove all packaging before operating table.

## Product use and features

No.	Instructions	
1.	Only lift using the table handles as shown with green arrow in photo. Lifting the table from any other place may damage the table.	XX

2.	Ensure battery is switched ON as indicated by green light. Battery is located underneath bottom frame as seen in photo. ON/OFF buttons are located approximately 9 cm from the side of the table. Press once to switch ON (green light blinks). Press and hold OFF button for three seconds to turn off. NB: Locate ON/OFF buttons using a mirror, phone camera, or by carefully lifting table slightly. If lifting, ensure castors are locked, use two or more persons to lift, and ONLY lift using the handles.	Light       Switch
3.	<ul> <li>The ALTRA is delivered fully tested and assembled.</li> <li>Charge battery before first use. First time charge takes at least 24 hours. Table may be used, and height adjusted during charging.</li> <li>Charging nightly is acceptable and does not damage battery or shorten battery life.</li> <li>Battery charge:</li> <li>Phase 1 – quick charge up to 80%</li> <li>Phase 2 – 10-hour slow charge of remaining 20%</li> <li>Table can be used after the quick charge, but battery must reach sufficient charge to do so.</li> </ul>	
4.	Two large foot controls are located on both sides of frame base for adjusting table height.	
5.	Stainless steel side rails (25 mm x 10 mm) on each side of the ALTRA can be used for accessories.	A PER
6.	Castors can be locked from rotation and swivel.	6

# Service and support

The ALTRA is made of durable materials and meant for long-term use. Most moving parts have maintenance-free brushings which need no extra lubrication.

Any required servicing should be performed by qualified personnel to ensure safety and maintenance of the product.

For further information, please contact iM3 or its official distributor for your country.

#### Warnings

The user of the ALTRA should be fully aware that it is a powerful machine with moving parts that may cause injury if precautions are not taken. The table is raised and lowered by an electrical actuator so care must be taken to ensure that hands, feet and other body parts are not in the path of the table movement.

Ensure the ALTRA is on a level surface and that the castors are locked before loading and unloading patients. Patients should be firmly secured when transporting.

Do not clean the table with running water as it may cause short circuit, damage the inner parts of the table or cause injury to you.

We hereby expressly disclaim and exclude liability for incidental, consequential, indirect or punitive damages, or financial loss to people or property resulting from any breach of warranty or from the improper operation of the ALTRA.

## Maintenance

For the long-term usage of the ALTRA, please follow the maintenance instructions below. For more information, please contact the iM3 official distributor.

- Do not use benzene, chlorides or other hazardous liquids or substances to clean the ALTRA.
- Use a soft cloth or vacuum cleaner to remove dust.
- Be careful to not damage or scratch the surface of the table.
- Ensure that there is nothing below or above the table when adjusting the height.
- Instruct cleaning personnel the correct way to move the ALTRA.
- Always use standard cleaning products meant for medical equipment.
- The inner construction of the ALTRA needs no daily maintenance.
- Never use sharp items or abrasive items to clean the ALTRA.
- Do not clean table or its parts with steam or in washing machine.

#### Warranty

- Warranty period is 24 months and is valued only in the official sales regions of iM3. Ask for updated information.

- All warranty-related repairs must be performed according to iM3's instructions and by iM3's authorised dealer.
- Warranty provisions do not apply to accessories or for normal wear.
- Warranty is not valid for damage resulting from negligence, unreasonable use, accident, exceeding weight limits or altering the product without prior iM3 authorisation.

## Technical data

- Measurements of work surface: 130 cm x 60 cm
- Adjustable height from 35 cm to 112 cm
- Total weight: 80 kg
- Maximum load capacity of the table: 150 kg
- Motors: 24 V, IP 54
- CE marked
- Manufactured by EQUA

CE