1M3° PIEZO P6 TIP WEAR GUIDE

Knowing when to replace your tips is essential as tip wear will significantly reduce scaler performance – as it alters the ultrasonic frequency.

Keep this Wear Guide handy so you can easily check before scaling.


What are the results of working with a worn tip?

- · Inefficient scaling
- Inferior tip performance
- · Discomfort for both vet and patient
- Poor misting of coolant (water) around the scaling tip




Replacement tips are available from iM3 online or your local iM3 distributor. Learn more about iM3 tips at www.im3vet.eu or www.im3vet.co.uk

- A. Align base of piece with the line ______
- **B.** Place piece over relevant image starting from base to tip
- **C.** Check the tip wear by seeing what coloured bar your tip aligns with

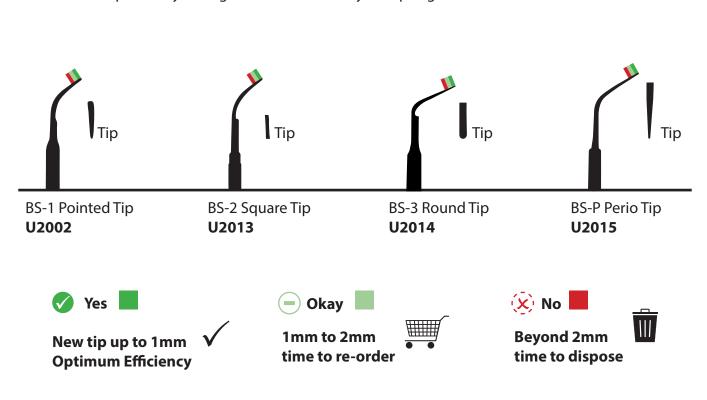



Tip worn beyond 2 mm please do NOT use

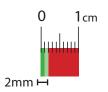

This scale is in cm and is intended to be printed at actual size on A4 paper.
Check the scale using a ruler to confirm it has printed at the correct size and adjust your printer settings if needed.

