

O-PHOSPHORIC ACID 36%

# Etchgel

Dental etching system

**iM3**<sup>®</sup>  
THE VETERINARY DENTAL COMPANY  
For veterinary use only

## CONTENTS

The set includes: a 6.5 g syringe filled with active substance (36% o-phosphoric acid) in the form of gel, and the iM3 Etchgel applicator tips.

## INDICATIONS

iM3 Etchgel is used to increase adhesion between teeth and composite restorations by dissolving the mineral component of teeth, thereby increasing the surface area for a bonding agent and preventing the occurrence of marginal fissures. It can be used for:

- self and light cured composites,
- fissure sealing,
- permanent and temporary stabilisation of teeth,
- fixed orthodontic appliances,
- preparation of metal surfaces as well as acrylic and composite restorations before using dual-cure composite resin cements.

## INSTRUCTIONS

Apply a thin layer of iM3 Etchgel on a carefully cleaned and dried surface. Leave for 15 seconds (s), then rinse with a strong stream of water. Dry thoroughly (with soft air pressure) until the enamel is matt-white, but to not desiccate tooth surface. Repeat if the enamel has not attained the matt effect or if contamination with saliva occurs. Do not etch for longer than 30 s.

## COMPOSITION

Active substance – o-phosphoric acid, water, gelling agent, dye.

## PRECAUTIONS

Use eye protection for both the patient and personnel. Avoid contact with skin, eyes and the soft tissues of the mouth. In case of such contact, rinse with plenty of water. In case of contact with eyes, seek immediate medical attention. In case of aspiration into the respiratory tract, seek immediate medical attention.

For use by veterinarians only.

## STORAGE

Store at a temperature between 3°C and 28°C (37°F and 82°F). If stored at a low temperature, bring back to room temperature before use. Keep out of reach of children. To prevent drying of iM3 Etchgel in the applicator, always retract the plunger of the syringe. Use in accordance with the manufacturer's instructions and principles of dentistry. Do not use after the expiry date. Instruct the patient on proper oral hygiene.

## WARRANTY

iM3 will replace products that have been proved to be defective or will refund the price of purchase. iM3 is not liable for any loss or damage caused by misuse or improper use of the product.

Instruction for use issued on 21/08/2019



Manufactured for

**iM3 THE VETERINARY DENTAL COMPANY**

USA (800) 664 6348 info@im3usa.com

Australia +61 2 9420 5766 sales@im3vet.com

Ireland +353 (0)1 6911277 sales@im3dental.com



**FOR VETERINARY USE ONLY**

**CE 0120**