

# OralVive™


(Physical barrier gel for topical application to teeth and gums)

For use in companion animals only; not for use in animals intended for human consumption.

**PRODUCT DESCRIPTION:** OralVive - is a 0.8% hyaluronic acid gel formulation (pH 6.5 – 7.5) supplied as 3ml of gel in a 3ml prefilled capped luer-lock syringe with 4 separate applicator tips per pack (**Fig. 1**).

**INTENDED USE:** OralVive™ hyaluronic acid gel adheres to oral mucosal tissue to act as a matrix for cellular infiltration to aid in tissue healing and a barrier to reduce irritation associated with oral surgery, periodontal injury or ulcerations.

**HANDLING:** Prefilled syringe is for **SINGLE USE** and unused contents should be discarded and not reused or stored.

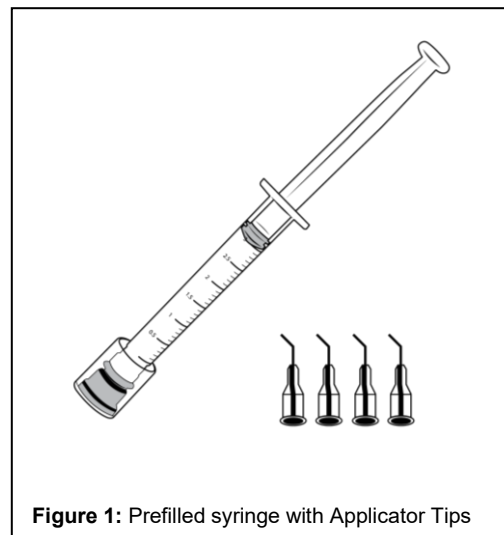

**Figure 1:** Prefilled syringe with Applicator Tips

**DIRECTIONS FOR USE:** OralVive™ gel is intended to be applied to slightly moist tissue. Excess fluid should be removed from tissue prior to applying OralVive™. After application, avoid lavaging or flushing gel from tissue. Remove the syringe from the box container, then remove the cap on the distal end of the syringe. If desired, the applicator tip can be secured to the luer-lock before applying the gel. Using manual pressure on the syringe plunger, the gel can be applied directly to periodontal tissue. If desired, the gel can be spread and distributed over the tissue with a gloved finger.

**CAUTION:** OralVive™ is an animal medical device for use in veterinary patients only. Product should be used by or under the supervision of a licensed veterinarian. This product is NOT for use in humans. Based on the clinical judgement of the veterinarian, appropriate debridement or lavage of periodontal and mucosal tissue prior to product application is recommended.

**CONTRAINDICATIONS:** Product use should be discontinued if the condition worsens. Product should not be used in veterinary patients with a known hypersensitivity to hyaluronic acid or any of its excipients.

**PRECAUTIONS/WARNINGS:** Inappropriate use may reduce performance of OralVive. **DO NOT** use if the package is opened or damaged. **DO NOT** autoclave products. Each syringe is for **SINGLE USE** only.

**SUPPLIED AS:** OralVive - is a 0.8% hyaluronic acid gel formulation (pH 6.5 – 7.5) supplied as 3ml of gel in a 3ml prefilled capped luer-lock syringe with 4 separate applicator tips per pack.

**List of ingredients:** Hyaluronic Acid, Poloxamer Gel, Glycerophosphocholine

**Storage condition:** Store below 25°C, do not refrigerate or freeze.

Keep out of sight and reach of children.

**Use by:** discard after 28 days of first opening

**Manufactured By:** Bova Specials UK Ltd  
Unit7-9 Gorst Road, Park Royal, UK

**Distributed by:** iM3 | The Global Name in Veterinary Dentistry (Ireland)

Contact Info:

EU/UK | +353 (0)1 691 1277 | [sales@im3dental.com](mailto:sales@im3dental.com)

AUS | +61 2 9420 5766 | [sales@im3vet.com](mailto:sales@im3vet.com)

USA | +1 800 664 6348 | [info@im3usa.com](mailto:info@im3usa.com)