

iM3® ShonaINSTRUCTIONS FOR USE AND
TECHNICAL MANUAL

Before we go any further, we want to say **thank you** for choosing iM3 Shona!

iM3 Shona has been developed by a dedicated team of engineers, veterinarians, and designers.

iM3 Shona Features:

- Made of electro-polished stainless steel.
- Middle motor column is anodised aluminium.
- Hand remote control easily adjusts height from 530 to 1100 mm.
- Strong actuator lifts up to 170 kg.
- Removable dental tray with round grid provides good access to patient and keeps patient's head dry during procedures.
- Innovative "wave breakers" within the tray make it easier to transport and empty liquids.
- X-frame base provides accessibility from all sides, even when seated.
- Four 100 mm quality German-made castors provide comfortable and smooth movement during patient transport. The Total Lock brake system locks both the rolling of the wheels and the rotation of the wheel fork.



Before first use

- Inspect packaging for any damage, take photos, and note all damages on the CMR before signing it. Later claims for transport damages are not accepted by transport companies.
- Please ensure adequately trained people unpack, install and power the table.
- Follow basic manual handling and safety guidelines when lifting the SHONA out of packaging. Do not hurt your back! Be safe!
- Make certain all moving parts are properly fixed, and all obstacles and packaging are removed prior to operating the table.


Mounting the round dental tray



iM3 Shona is delivered fully tested and assembled. The round dental tray (sink) attaches easily to the bracket at the end of the table.

Code	Accessory name / mounting instruction	Accessory name / photo
TB0002	<p>Dental Tray</p> <p>Remove the protective packing material. The dental end connection bracket is pre-installed and can be removed if needed.</p> <p>Attach the aluminium brackets of the dental tray (1) to the rail on the end of the table (2) ensuring the tray is centered.</p>	<p>Dental Tray</p>
TB0005	<p>Table Extension</p> <p>Remove the rail from the end of the table by opening the black handscrews (anti-clockwise) which are located under the rail (1) and pull rail out (2).</p> <p>Slide the hollow square tubes of the extension table into the larger hollow square tubes under the tabletop. Push in completely (3) and tighten securely using the black knobs under the tabletop (1).</p>	<p>Table Extension (30 cm)</p>
Standard	<p>Tie Down Cleats</p> <p>Located under the edge of the tabletop to secure the patient.</p>	<p>Tie Down Cleats</p>

Hand Remote Control

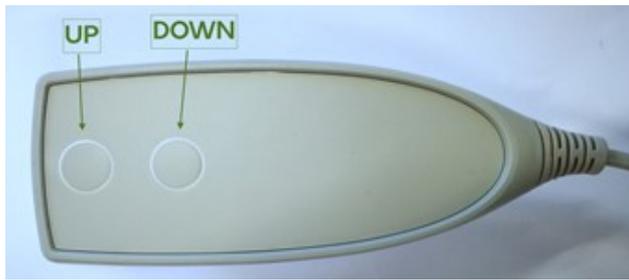

The height of iM3 Shona is adjusted by a wired hand control. The upper button raises the table and lower button lowers the table.

Brakes and castors

The castors are designed for safe and comfortable movement. The castors are fitted with brakes that lock the wheels and the forks, preventing rotation. **NOTE: Always lock all four castors when the table is in use.**



Service and Support

- The product is made of durable hard-wearing materials.
- The service routine must be performed by qualified personnel to ensure the safety and performance of the product.
- For further information, please contact iM3 or its official distributor in your country.

Maintaince, Do's and Dont's

For the long-term use of iM3 Shona and its accessories, please follow the maintaince instructions below. If you have questions or concerns, please contact iM3 or an official distributor.

- Never use benzenes, chlorides or other hazardous liquids or substances for cleaning the table or its accessories.
- Always use standard cleaning products meant for medical equipment and/or stainless steel surfaces.
- For dusting, gently use a vacuum cleaner or soft cloth taking care to not damage or scratch the surfaces.
- Always keep the table clear of obstacles when moving to avoid collision.
- Do not place the remote control into water or dental tray.
- Because damage is commonly inflicted by cleaning personnel, please provide them with proper instructions.
- The inner workings of iM3 Shona do not need daily maintenance.
- Never use sharp instruments or coarse scrubbers to clean the surfaces, instead let the cleaning agent do the job.
- Do not contaminate the stainless steel surfaces with iron dust or sparks, as this will cause surface corrosion.
- Do not clean the produce with the same cloth that was used to clean iron dust or other iron rich surfaces as it will cause surface corrosion.
- Except the dental tray, do not steam clean the table or wash parts in the washing machine.

Warranty

- Warranty period is 24 months and is valued only in the official sales regions of iM3. For updated information, please see www.im3vet.com or contact sales@im3dental.com.
- All warranty related repair works must be performed according to iM3 approved instructions and by iM3 authorized dealer.
- Warranty provisions exclude accessories and normal wear and tear.
- Warranty provisions do not apply in cases of misuse or physical damages of the product.

Technical Data

- Measurements of work surface: 120 x 60 cm
- Adjustable height from 53 to 110 cm
- Total weight: 50 kg, Load capacity: 170 kg
- Motors, 2000N, 24 V, IP 54
- CE marked



Manufactured by iM3 and EQUA